

LBF® Sterile No Sting Barrier Film Spray (30ml and 50ml)

Product Description

LBF Sterile No Sting Barrier Film Spray contains a no sting healthcare-grade silicone fluid that forms a transparent film on the skin and that acts as a protective barrier against damage from bodily fluids and waste and skin stripping caused by repeated application and/or removal of adhesives.

Indications for use

LBF Sterile No Sting Barrier Film Spray is indicated for the protection of intact and injured skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives. The product may be used on adults and on children.

Contraindications

Do not use on mucosal membranes and broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near eyes, ears, nose, or mouth. Avoid direct inhalation or use in enclosed spaces. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of LBF Sterile. May not be suitable for use on babies under 30 days old.

Instructions for use

1. Skin should be clean and dry prior to application.
2. Spray a uniform coating to the area of concern.
3. Dries in seconds. Allow LBF Sterile No Sting Barrier Film Spray to evaporate completely before covering. Once dried, it will not affect any adhesion of further appliances.
4. If LBF Sterile No Sting Barrier Film Spray is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.
5. When used under stoma pouches, adhesive tapes or dressings, reapplication is necessary with every change of adhesive appliance as the barrier is removed by the adhesive.
6. When used as a protectant against corrosive bodily fluids (including urine and faeces) and no adhesive products are applied to the skin, reapplication is recommended every 24-48 hours, depending on the frequency of cleaning. In extreme cases (e.g. constant diarrhoeal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
7. If desired, the film can be removed by using most medical adhesive removers.

Ingredients











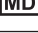


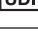


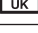

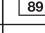
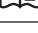

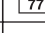
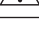
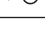
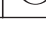
Siloxanes.

Storage and Shelf Life

Store away from direct sunlight at ambient temperature and humidity. For shelf life, refer to the expiry date at the base of the can.

Warning

Flammable aerosol. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition source. Do not pierce or burn, even after use. Do not expose to temperatures above 50°C/122°F. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the can is damaged. Contact CliniMed in the event that the can is received damaged before use. Recycle empty cans in accordance with local regulations.

	Manufacturer		Single Sterile Barrier System		Keep away from Sunlight
	EC Representative		Single Sterile Barrier System with Protective Packaging Outside		Single Patient Multiple use
	Catalogue Number				Do not use if packaging is damaged and consult instructions for use
	Batch Code		Sterilised by Irradiation		
	Medical Device		Use-by Date		Do Not Re-sterilise
	Unique Device Identifier		Flammable		Upper Temperature Limit
	Country of Manufacture		Recyclable (Aluminium)		Brimful Volume
	Instructions for Use		Recycling (Paper & Cardboard)		Brimful Volume
	Caution/Warning		Not Made with Natural Rubber Latex		Estimate

Product Code	Description	No. Per Box
3827	LBF Sterile No Sting Barrier Film Spray 30ml	1
3826	LBF Sterile No Sting Barrier Film Spray 50ml	1

CliniMed

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.
Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk
CliniMed® and LBF® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd, Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022
PID 10882



1639 0120