

NO STING

Appeel[®]

STERILE

**No Sting Medical Adhesive
Remover Spray**

REF 35100

CliniMed[®]

Appeel[®] Sterile No Sting

Medical Adhesive Remover Spray

Product Description

Appeel Sterile No Sting Medical Adhesive Remover Spray is a no sting healthcare-grade silicone fluid which aids the removal of stoma pouches and other adhesive appliances from the skin. Use of the product helps to reduce skin stripping and peri-wound trauma and pain caused by the removal of adhesive appliances. Appeel Sterile No Sting Medical Adhesive Remover Spray helps maintain skin integrity whilst assisting quick and easy appliance removal. It is fast drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use.

Indications for use

Appeel Sterile No Sting Medical Adhesive Remover Spray is indicated for use on intact and injured skin, facilitating the quick and easy removal of stoma bags and other adhesive appliances. This product may be used on adults and children.

Contraindications

Appeel Sterile No Sting Medical Adhesive Remover Spray is not intended for direct use on broken skin. If in doubt, patch test a small area prior to use.

Caution

Use only as directed. For external use only. Allow product to evaporate completely before covering. Sterile device. Do not use near eyes, ears, nose, or mouth. Avoid direct inhalation or use in enclosed spaces. May not be suitable for use on babies under 30 days old.

Instructions For Use

1. Skin should be clean and dry prior to application.
2. Spray Appeel Sterile No Sting Medical Adhesive Remover Spray around the edge of the adhesive appliance to reduce skin trauma.
3. Wait a few seconds, then gently peel adhesive appliance away from the skin using additional spray to ease removal, if necessary.
4. Dries in seconds. Allow to evaporate completely before covering. Once dried Appeel Sterile No Sting Medical Adhesive Remover Spray will not affect adhesion of further appliances.






















Appeel® Sterile No Sting

Medical Adhesive Remover Spray

Warning: Flammable aerosol. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition source. Do not pierce or burn, even after use. Do not expose to temperatures above 50°C/122°F. Keep out of reach of children. Do not use in cases where there is a known sensitivity to any of the ingredients. Discontinue if irritation occurs and report to the manufacturer quoting the LOT number. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. If eye contact/ingestion occurs, seek immediate medical attention. Do not use if the can is damaged. Contact CliniMed in the event that the can is received damaged before use. Recycle empty cans in accordance with local regulations.

Ingredients: Siloxanes

Storage and Shelf Life: Store away from direct sunlight at ambient temperature. For shelf life, refer to the expiry date on the base of the can.

	Manufacturer		Caution/Warning		Single Patient Multiple use
	EC Representative		Single Sterile Barrier System		Do Not Re-sterilise
	Catalogue Number		Sterilised by Irradiation		Upper Temperature Limit
	Batch Code		Use-by Date		Brimful Volume
	Medical Device		Flammable		Estimate
	Unique Device Identifier		Recyclable (Aluminium)		
	Country of Manufacture		Not Made with Natural Rubber Latex		
	Instructions for Use		Keep away from Sunlight		


Product Code	Description
35100	Appeel Sterile No Sting Medical Adhesive Remover Spray

CliniMed®

CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY, UK.

Tel 01628 850100 Email info@clinimed.co.uk Web www.clinimed.co.uk

CliniMed® and Appeel® are registered trademarks of CliniMed (Holdings) Limited. ©CliniMed 2022.

 Advena Ltd. Tower Business Centre, 2nd Floor, Tower Street, Swatar, BKR 4013 Malta

Rev. 12/2022
PID 10867


1639 0120